INSTRUCTION MANUAL

Volt CPAP Battery Pack

PLEASE READ THIS SHEET CAREFULLY TO ENSURE SAFE USAGE AND MAINTENANCE

The Viverity™ Volt CPAP Battery Pack provides you with a simple solution at just over 2 lbs, including carrying case. It comes complete with a large capacity battery, AC charger and handy carrying case. The carrying case has a handle, shoulder strap and four external zipper storage pouches to give you a wide range of storage capabilities. This unit will provide power to your CPAP unit for up to 14 hours* of operation.

NOTE

The Volt CPAP Battery Pack requires a manufacturer's DC auto adapter for your individual CPAP/Bi-PAP unit (purchased separately). See your Homecare Provider or the original manufacturer of your CPAP unit.

Use:

1. Remove the charger from the case and plug the male 12 volt end of the charger into the female 12 volt outlet on the CPAP battery.
2. Charge the battery until the light on the charger turns green or a minimum of 6 hours on the first charge. A red light on the charger indicates that the battery is still charging.
3. When fully charged, use the manufacturer's DC cord (not included) to connect your CPAP to the outlet on the Volt CPAP Battery Pack.

NOTE

Most heated humidifiers will disable while on battery operation due to excessive battery consumption. The humidifier can be used in cold passover mode instead while battery is in use.

Always:

- Always charge the battery fully prior to use. Only use the provided charger for charging your Volt CPAP Battery Pack.
- Always use your Volt CPAP Battery Pack and charger in well ventilated areas.
- Always allow the battery to cool to room temperature prior to charging, in case it has been heated while stored in a car, truck, etc.
- Always disconnect the CPAP unit from the battery when not in use. The displays on the units consume power, shortening your usage time.

- Always remove the Volt CPAP Battery Pack from the case when charging. It is normal for it to feel warm when charging a very low battery.
- Always stop using the battery if a leak is detected or if the unit becomes extremely hot.

Never:

- Never try to open or pierce the wrap around the batteries to service it.
- Never try to service the battery in any way. It is factory sealed and changes may cause damage to the unit or your CPAP.
- Never replace the batteries individually or use in conjunction with another charger, as damage can occur.
- Never submerge/allow batteries to become wet.
- Never subject the Volt CPAP Battery Pack to temperatures over 140° F.
- Never allow children to play with your Volt CPAP Battery Pack, as it poses a choking and electrical hazard.
- Never throw away or incinerate your Volt CPAP Battery Pack. Please dispose of the battery according to local waste guidelines for recycling.
- Never place the battery in a microwave or pressurized container.
- Never place the Volt CPAP Battery Pack in checked luggage of any aircraft. It is safe to include with carry-on bags.

Battery Type	Li-Ion 18650
Battery Pack	14.8 VAC
Charger	15.6 VDC-3A OUTPUT, 100-240 VAC INPUT, with Charge Indicator and Male 12 Volt Auto Connector
Weight	2.4 lbs.
Rated Capacity	7,800 mAh
Minimum Capacity	7,500 mAh
Rated Energy	115.44 Watts
Loaded Voltage	14.4 Volts
Cut Off Voltage	11.00 Volts
Charge Voltage	15.6 Volts
Maximum Charge Current	3,900 mA
Maximum Discharge Current	1,560 mA
Operating Temperature	32°F – 113°F (0°C – 45°C)
Storage Temperature	32°F – 113°F (0°C – 45°C)

Operation

Hours*	cm H ₂ O
(up to) 9	15
(up to) 14	10

Roscoe Medical, Inc.

Strongsville, OH 44149
 800.376.7263 • roscoemedical.com
 A division of Compass Health Brands
 Made in China ©2015

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